


aldosterone hypoglycemia mineralocorticoid permanent alertness  
 blood test exhaustion allergies slurred speech anxiety neurotransmitters  
 apathy adrenal fatigue disinterest body aches  
 autoimmune diseases adrenal energy loss lightheadedness  
 fight or flight hormonal depletion difficulty getting up late night energy  
 caffeine saliva test kidney tired shaking glands foggy  
 low energy adrenal energy loss  
 norepinephrine sleep low sex drive kidney  
 diet chronic stress inflammation hair loss poor circulation  
 irritability dizziness stress-related loss of muscle tone  
 blood sugar insufficient sleep vitamin c lower back pain  
 frequent urination insomnia hormones



**Cortisol Quantitative Test**

*Proprietary Rapid Cortisol  
“Sandwich” Quantitative Whole Blood Test*

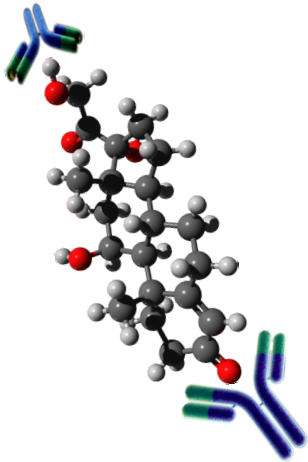

**GAME-CHANGER  
IMMUNO-SANDWICH TEST  
TECHNOLOGY**



**RapiRead**  
CUBE READER




Quantify Cortisol  
in just 3 seconds with  
World's Smallest Reader: **RapiRead**




## EASY TO USE TEST PROCEDURE WITH UniSampler™ DEVICE


(Consult Instructions for Use for complete procedure)

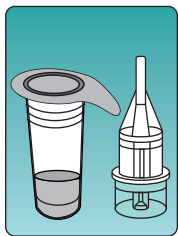


1. UniSampler™ Device

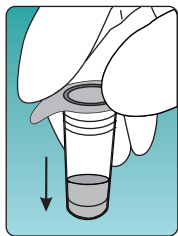


2. Tap the Buffer Tube.

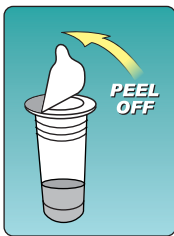


3. Peel Off Aluminum Foil.

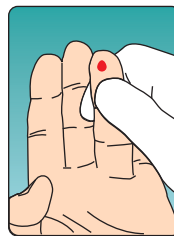


4. Use a Lancet to draw Finger-prick Blood.

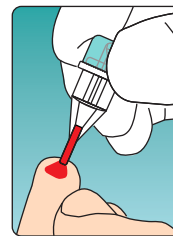


5. Gently touch the tip of Blood Collector to Blood droplet to fill 10µl blood.

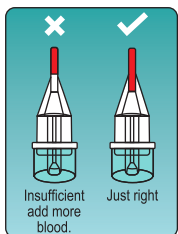


6. Check the level of Blood.

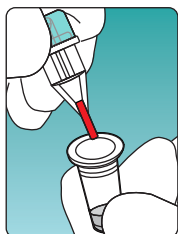


7. Insert Blood Collector into Tube and push firmly.

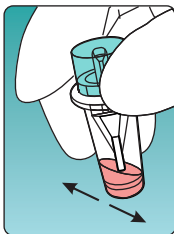

8. Shake vigorously and mix completely.

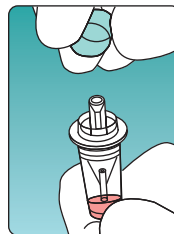


9. Remove Cap.

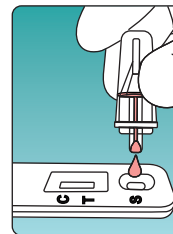

10. Add 3 drops to Sample well (S) and read result at 15 minutes using RapiRead™ CUBE Reader.

## CLEAR RESULT INTERPRETATION

(Consult Instructions for Use for Digital Result Interpretations using RapiRead™ Cube Reader)

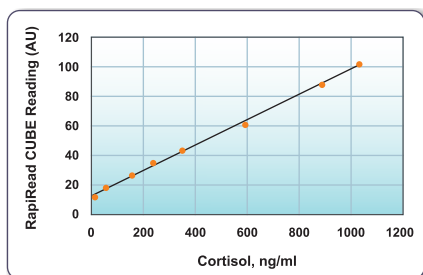

### STANDARD CURVE

The Cortisol values of serum calibrators are assigned by using a competitive antibody-binding assay and Standard Reference Material 921a (NIST).

## Product Features:

- Finger-prick Blood (10µl) or Serum samples (5µl)
- No pre-incubation required
- Quick Quantitative results in just 15 minutes
- Sensitivity: 30 ng/ml
- Specificity: 100%
- Detection Range: 30 ng/ml – 1000 ng/ml
- Traceability: Standard Reference Material 921a (NIST)
- Simple, Convenient and Easy to interpret
- Accurate and Reliable
- Lab Grade Cortisol Test at any Point-Of-Care setting
- Diagnosis & treatment made easier
- Immediate Consultation



Affimedix, Inc.  
3556 Investment Boulevard  
Hayward, California 94545, USA  
Tel: (510) 398-8111  
Fax: (510) 398-8132  
Email: info@affimedix.com  
www.affimedix.com  
www.Rapi-D.com

EC REP

Qarad EC-REP BV  
Pas 257  
2440 Geel  
Belgium  
www.qarad.com



Anytime, Anywhere!